



**PRIMARY TOTAL KNEE
REPLACEMENT**

01

General presentation

- p.3 - p.4

02

List of reference

- p.5 - p.9 -

03

Technical information

- p.10 - p.20 -

04

Identification

- p.22 - p.32

01

General presentation

- ∅ Femoral Components, tibial baseplates, fixed bearing inserts and patellar implants are class III sterile medical devices in accordance with rule 8 annex VIII of regulation (EU) 2017/745.
- ∅ Tibial augments and extension stems are class IIb sterile medical devices in accordance with rule 8 annex VIII of regulation (EU) 2017/745.
- ∅ This primary total knee replacement (TKR) with a fixed bearing insert is intended for the replacement of the total knee joint in order to reduce pain and restore knee function in comparison with pre-operative status.
- ∅ It includes 4 parts: the femoral component which replaces the distal part of the femur, the tibial baseplate which replaces the proximal part of the tibia, the insert which ensures the connection between the femur and the tibia, and the patellar implant which replaces the articular surface of the patella.
- ∅ The femoral component is available in 9 sizes (left and right version). It is made of cobalt chrome alloy (CoCr). It is available in cemented and cementless version (with hydroxyapatite + titanium spray double coating).
- ∅ The tibial baseplate is available in 9 sizes. All surfaces in contact with the insert are mirror polished. The insert is guided in the posterior part of the baseplate and is fixed on its anterior edge. It is made of cobalt chrome alloy (CoCr) and is available in cemented or cementless version (with hydroxyapatite + titanium spray double coating).
- ∅ The insert connects the femur and the tibia, providing mobility and stability of one in relation to the other. It is fixed onto the tibial baseplate. It is available in 9 sizes and 6 thicknesses. It is made of ultra high molecular weight polyethylene (UHMWPE).


01 General presentation

Patellar implants are available in 2 versions: resurfacing or inset patella


Patellar implants are made of ultra high molecular weight polyethylene (UHMWPE)

Resurfacing patellar implants are cemented. They are available in 3 versions:

- ⌋ The cemented resurfacing patellar implant is available in 4 diameters (30, 33, 36 and 39mm). The distance between the pegs changes according to the size.

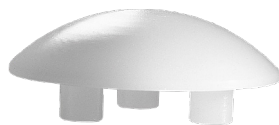

- ⌋ The cemented resurfacing patellar implant NM is available in 4 diameters (30, 33, 36 and 39mm). The distance between the pegs is constant for all sizes.

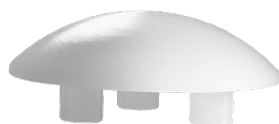

- ⌋ The cemented asymmetric congruent resurfacing patellar implant is available in 5 sizes (26x33, 29x36, 32x39, 35x42 and 38x45mm). The distance between the pegs is constant for all sizes.



Inset patellar implant is available in a cemented version:

- ⌋ The cemented inset patellar implant is available in 3 diameters (23, 26 and 29mm).



In cases of tibial revision or complex case, tibial augments and tibial extension stems are available.



ANATOMIC Femoral Component

ANATOMIC posterior stabilized femoral component Cementless HA coated	Reference
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 0 Right	1-0204300
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 1 Right	1-0204301
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 2 Right	1-0204302
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 3 Right	1-0204303
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 4 Right	1-0204304
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 5 Right	1-0204305
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 6 Right	1-0204306
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 7 Right	1-0204307
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 8 Right	1-0204308
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 0 Left	1-0204400
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 1 Left	1-0204401
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 2 Left	1-0204402
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 3 Left	1-0204403
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 4 Left	1-0204404
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 5 Left	1-0204405
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 6 Left	1-0204406
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 7 Left	1-0204407
ANATOMIC posterior stabilized femoral component Cementless HA coated Size 8 Left	1-0204408

ANATOMIC posterior stabilized femoral component Cemented	Reference
ANATOMIC posterior stabilized femoral component Cemented Size 0 Right	1-0204500
ANATOMIC posterior stabilized femoral component Cemented Size 1 Right	1-0204501
ANATOMIC posterior stabilized femoral component Cemented Size 2 Right	1-0204502
ANATOMIC posterior stabilized femoral component Cemented Size 3 Right	1-0204503
ANATOMIC posterior stabilized femoral component Cemented Size 4 Right	1-0204504
ANATOMIC posterior stabilized femoral component Cemented Size 5 Right	1-0204505
ANATOMIC posterior stabilized femoral component Cemented Size 6 Right	1-0204506
ANATOMIC posterior stabilized femoral component Cemented Size 7 Right	1-0204507
ANATOMIC posterior stabilized femoral component Cemented Size 8 Right	1-0204508
ANATOMIC posterior stabilized femoral component Cemented Size 0 Left	1-0204600
ANATOMIC posterior stabilized femoral component Cemented Size 1 Left	1-0204601
ANATOMIC posterior stabilized femoral component Cemented Size 2 Left	1-0204602
ANATOMIC posterior stabilized femoral component Cemented Size 3 Left	1-0204603
ANATOMIC posterior stabilized femoral component Cemented Size 4 Left	1-0204604
ANATOMIC posterior stabilized femoral component Cemented Size 5 Left	1-0204605
ANATOMIC posterior stabilized femoral component Cemented Size 6 Left	1-0204606
ANATOMIC posterior stabilized femoral component Cemented Size 7 Left	1-0204607
ANATOMIC posterior stabilized femoral component Cemented Size 8 Left	1-0204608

*Check availability in your country

ANATOMIC Tibial Base Plate

ANATOMIC tibial base plate for fixed bearing insert	Reference
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 0	1-0204800
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 1	1-0204801
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 2	1-0204802
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 3	1-0204803
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 4	1-0204804
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 5	1-0204805
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 6	1-0204806
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 7	1-0204807
ANATOMIC tibial base plate for fixed bearing insert Cementless HA coated Size 8	1-0204808
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 0	1-0204900
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 1	1-0204901
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 2	1-0204902
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 3	1-0204903
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 4	1-0204904
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 5	1-0204905
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 6	1-0204906
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 7	1-0204907
ANATOMIC tibial base plate for fixed bearing insert Cemented Size 8	1-0204908

ANATOMIC Fixed Bearing Insert

ANATOMIC fixed bearing insert	Reference
ANATOMIC fixed bearing insert Size 0 Thickness 10 mm	1-0204701
ANATOMIC fixed bearing insert Size 0 Thickness 12 mm	1-0204702
ANATOMIC fixed bearing insert Size 0 Thickness 14 mm	1-0204703
ANATOMIC fixed bearing insert Size 0 Thickness 16 mm	1-0204704
ANATOMIC fixed bearing insert Size 0 Thickness 18 mm	1-0204705
ANATOMIC fixed bearing insert Size 0 Thickness 20 mm	1-0204706
ANATOMIC fixed bearing insert Size 1 Thickness 10 mm	1-0204710
ANATOMIC fixed bearing insert Size 1 Thickness 12 mm	1-0204711
ANATOMIC fixed bearing insert Size 1 Thickness 14 mm	1-0204712
ANATOMIC fixed bearing insert Size 1 Thickness 16 mm	1-0204713
ANATOMIC fixed bearing insert Size 1 Thickness 18 mm	1-0204714
ANATOMIC fixed bearing insert Size 1 Thickness 20 mm	1-0204715
ANATOMIC fixed bearing insert Size 2 Thickness 10 mm	1-0204720
ANATOMIC fixed bearing insert Size 2 Thickness 12 mm	1-0204721
ANATOMIC fixed bearing insert Size 2 Thickness 14 mm	1-0204722
ANATOMIC fixed bearing insert Size 2 Thickness 16 mm	1-0204723
ANATOMIC fixed bearing insert Size 2 Thickness 18 mm	1-0204724
ANATOMIC fixed bearing insert Size 2 Thickness 20 mm	1-0204725

ANATOMIC Fixed Bearing Insert

ANATOMIC fixed bearing insert	Reference
ANATOMIC fixed bearing insert Size 3 Thickness 10 mm	1-0204730
ANATOMIC fixed bearing insert Size 3 Thickness 12 mm	1-0204731
ANATOMIC fixed bearing insert Size 3 Thickness 14 mm	1-0204732
ANATOMIC fixed bearing insert Size 3 Thickness 16 mm	1-0204733
ANATOMIC fixed bearing insert Size 3 Thickness 18 mm	1-0204734
ANATOMIC fixed bearing insert Size 3 Thickness 20 mm	1-0204735
ANATOMIC fixed bearing insert Size 4 Thickness 10 mm	1-0204740
ANATOMIC fixed bearing insert Size 4 Thickness 12 mm	1-0204741
ANATOMIC fixed bearing insert Size 4 Thickness 14 mm	1-0204742
ANATOMIC fixed bearing insert Size 4 Thickness 16 mm	1-0204743
ANATOMIC fixed bearing insert Size 4 Thickness 18 mm	1-0204744
ANATOMIC fixed bearing insert Size 4 Thickness 20 mm	1-0204745
ANATOMIC fixed bearing insert Size 5 Thickness 10 mm	1-0204750
ANATOMIC fixed bearing insert Size 5 Thickness 12 mm	1-0204751
ANATOMIC fixed bearing insert Size 5 Thickness 14 mm	1-0204752
ANATOMIC fixed bearing insert Size 5 Thickness 16 mm	1-0204753
ANATOMIC fixed bearing insert Size 5 Thickness 18 mm	1-0204754
ANATOMIC fixed bearing insert Size 5 Thickness 20 mm	1-0204755
ANATOMIC fixed bearing insert Size 6 Thickness 10 mm	1-0204760
ANATOMIC fixed bearing insert Size 6 Thickness 12 mm	1-0204761
ANATOMIC fixed bearing insert Size 6 Thickness 14 mm	1-0204762
ANATOMIC fixed bearing insert Size 6 Thickness 16 mm	1-0204763
ANATOMIC fixed bearing insert Size 6 Thickness 18 mm	1-0204764
ANATOMIC fixed bearing insert Size 6 Thickness 20 mm	1-0204765
ANATOMIC fixed bearing insert Size 7 Thickness 10 mm	1-0204770
ANATOMIC fixed bearing insert Size 7 Thickness 12 mm	1-0204771
ANATOMIC fixed bearing insert Size 7 Thickness 14 mm	1-0204772
ANATOMIC fixed bearing insert Size 7 Thickness 16 mm	1-0204773
ANATOMIC fixed bearing insert Size 7 Thickness 18 mm	1-0204774
ANATOMIC fixed bearing insert Size 7 Thickness 20 mm	1-0204775
ANATOMIC fixed bearing insert Size 8 Thickness 10 mm	1-0204780
ANATOMIC fixed bearing insert Size 8 Thickness 12 mm	1-0204781
ANATOMIC fixed bearing insert Size 8 Thickness 14 mm	1-0204782
ANATOMIC fixed bearing insert Size 8 Thickness 16 mm	1-0204783
ANATOMIC fixed bearing insert Size 8 Thickness 18 mm	1-0204784
ANATOMIC fixed bearing insert Size 8 Thickness 20 mm	1-0204785

02

List of references*

Patellar Implants

Cemented Resurfacing Patellar implant	Reference
Cemented resurfacing patellar implant - Ø 30 mm	1-0200830
Cemented resurfacing patellar implant - Ø 33 mm	1-0200833
Cemented resurfacing patellar implant - Ø 36 mm	1-0200836
Cemented resurfacing patellar implant - Ø 39 mm	1-0200839

Cemented Resurfacing Patellar implant NM	Reference
Cemented Resurfacing Patellar implant NM Ø 30 mm	1-0201430
Cemented Resurfacing Patellar implant NM Ø 33 mm	1-0201433
Cemented Resurfacing Patellar implant NM Ø 36 mm	1-0201436
Cemented Resurfacing Patellar implant NM Ø 39 mm	1-0201439

Cemented inset patellar implant	Reference
Cemented inset patellar implant - Ø 23 mm	1-0200923
Cemented inset patellar implant - Ø 26 mm	1-0200026
Cemented inset patellar implant - Ø 29 mm	1-0200029

Cemented Asymmetric Congruent Resurfacing Patellar Implant	Reference
Cemented Asymmetric Congruent Resurfacing Patellar Implant - 26x33 mm	1-0201833
Cemented Asymmetric Congruent Resurfacing Patellar Implant - 29x36 mm	1-0201836
Cemented Asymmetric Congruent Resurfacing Patellar Implant - 32x39 mm	1-0201839
Cemented Asymmetric Congruent Resurfacing Patellar Implant - 35x42 mm	1-0201842
Cemented Asymmetric Congruent Resurfacing Patellar Implant - 38x45 mm	1-0201845

OPTIONS

Stems, Augments

Extension stem for Total Knee Prosthesis – Cemented	Reference
Extension stem for Total Knee Prosthesis – Cemented - Ø10 mm length 75mm	1-0200710
Extension stem for Total Knee Prosthesis – Cemented - Ø10 mm length 100 mm	1-0200711
Extension stem for Total Knee Prosthesis – Cemented - Ø10 mm length 150 mm	1-0200712
Extension stem for Total Knee Prosthesis – Cemented - Ø12 mm length 75mm	1-0200720
Extension stem for Total Knee Prosthesis – Cemented - Ø12 mm length 100 mm	1-0200721
Extension stem for Total Knee Prosthesis – Cemented - Ø12 mm length 150 mm	1-0200722
Extension stem for Total Knee Prosthesis – Cemented - Ø14 mm length 75mm	1-0200730
Extension stem for Total Knee Prosthesis – Cemented - Ø14 mm length 100 mm	1-0200731
Extension stem for Total Knee Prosthesis – Cemented - Ø14 mm length 150 mm	1-0200732
Extension stem for Total Knee Prosthesis – Cemented - Ø16 mm length 100 mm	1-0200741
Extension stem for Total Knee Prosthesis – Cemented - Ø16 mm length 150 mm	1-0200742

ANATOMIC Tibial Augment	Reference
ANATOMIC Tibial Augment Size 0/1/2 Thickness 5 mm	1-0210610
ANATOMIC Tibial Augment Size 3/4/5 Thickness 5 mm	1-0210613
ANATOMIC Tibial Augment Size 6/7/8 Thickness 5 mm	1-0210616
ANATOMIC Tibial Augment Size 0/1/2 Thickness 10 mm RM/LL	1-0210620
ANATOMIC Tibial Augment Size 3/4/5 Thickness 10 mm RM/LL	1-0210623
ANATOMIC Tibial Augment Size 6/7/8 Thickness 10 mm RM/LL	1-0210626
ANATOMIC Tibial Augment Size 0/1/2 Thickness 10 mm RL/LM	1-0210630
ANATOMIC Tibial Augment Size 3/4/5 Thickness 10 mm RL/LM	1-0210633
ANATOMIC Tibial Augment Size 6/7/8 Thickness 10 mm RL/LM	1-0210636
ANATOMIC Tibial Augment Size 0/1/2 Thickness 15 mm RM/LL	1-0210640
ANATOMIC Tibial Augment Size 3/4/5 Thickness 15 mm RM/LL	1-0210643
ANATOMIC Tibial Augment Size 6/7/8 Thickness 15 mm RM/LL	1-0210646
ANATOMIC Tibial Augment Size 0/1/2 Thickness 15 mm RL/LM	1-0210650
ANATOMIC Tibial Augment Size 3/4/5 Thickness 15 mm RL/LM	1-0210653
ANATOMIC Tibial Augment Size 6/7/8 Thickness 15 mm RL/LM	1-0210656

*Check availability in your country

03

Technical information

ANATOMIC posterior stabilized femoral component cementless - HA coated



Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.



Presence of latex in the product's composition: None

Material:

Implantable chrome cobalt alloy according to standard ISO 5832-4
Hydroxyapatite (HA) coating according to standards ISO 13779-2, ISO 13779-6 and ASTM F1185 +
Titanium undercoating according to standards ISO 13179-1

Manufacturing process:

Casting
Inspection
Machining
Polishing
Engraving
Inspection
Passivation
Coating
Cleaning
Inspection
Final cleaning
Packaging
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2013

03

Technical information

ANATOMIC posterior stabilized femoral component – cemented

Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.

Presence of latex in the product's composition: None

Material:

Implantable chrome cobalt alloy according to standard ISO 5832-4

Manufacturing process:

Casting
Inspection
Machining
Grit blasting
Polishing
Engraving
Inspection
Passivation
Final cleaning
Packaging
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2013

03

Technical information

ANATOMIC tibial base plate for fixed bearing insert - cementless HA-coated



Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.



Presence of latex in the product's composition: None

Material:

Implantable chrome cobalt alloy according to standard ISO 5832-4
Implantable stainless steel (M30NW) according to standard ISO 5832-9 (standard keel)
Hydroxyapatite (HA) coating according to standards ISO 13779-2, ISO 13779-6 and ASTM F1185 +
Titanium undercoating according to standards ISO 13179-1

Manufacturing process:

Casting
Control
Machining
Polishing
Engraving
Inspection
Passivation
Coating
Cleaning
Gathering
Inspection
Final cleaning
Packaging
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2013

03

Technical information

ANATOMIC tibial base plate for fixed bearing insert – cemented

Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.

Presence of latex in the product's composition: None

Material:

Implantable chrome cobalt alloy according to standard ISO 5832-4

Implantable stainless steel (M30NW) according to standard ISO 5832-9 (standard keel)

Manufacturing process:

Casting
Inspection
Machining
Grit blasting
Polishing
Engraving
Inspection
Passivation
Gathering
Final cleaning
Packaging
Packing
Sterilization
Release

CE Mark :

Notified body: BSI Group The Netherlands B.V. (CE2797)

Date EC marking first applied: 2013

03

Technical information

ANATOMIC fixed bearing insert posterior stabilized

Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.

Presence of latex in the product's composition: None

Material:

Implantable Ultra High Molecular Weight Polyethylene (UHMWPE) according to standard ISO 5834-1 and ISO 5834-2

Manufacturing process:

Machining
Engraving
Inspection
Final cleaning
Packaging
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2013

03

Technical information

Cemented Resurfacing Patellar Implant

Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.

Presence of latex in the product's composition: None

Material:

Implantable Ultra High Molecular Weight Polyethylene (UHMWPE) according to standard ISO 5834-1 and ISO 5834-2

Manufacturing process:

Machining
Engraving
Control
Internal labels printing
Cleaning
Packaging Clean Room
Boxing
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2003

03

Technical information

Cemented Resurfacing Patellar Implant NM



Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.



Presence of latex in the product's composition: None

Material:

Implantable Ultra High Molecular Weight Polyethylene (UHMWPE) according to standard ISO 5834-1 and ISO 5834-2

Manufacturing process:

Machining
Engraving
Control
Internal labels printing
Cleaning
Packaging Clean Room
Boxing
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2016

03

Technical information

Cemented inset patellar implant

Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.

Presence of latex in the product's composition: None

Material:

Implantable Ultra High Molecular Weight Polyethylene (UHMWPE) according to standard ISO 5834-1 and ISO 5834-2

Manufacturing process:

Machining
Engraving
Control
Internal labels printing
Cleaning
Packaging Clean Room
Boxing
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2003

03

Technical information

Cemented Asymmetric Congruent Resurfacing Patellar Implant



Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.



Presence of latex in the product's composition: None

Material:

Implantable Ultra High Molecular Weight Polyethylene (UHMWPE) according to standard ISO 5834-1 and ISO 5834-2

Manufacturing process:

Machining
Engraving
Control
Internal labels printing
Cleaning
Packaging Clean Room
Boxing
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2016

03

Technical information

ANATOMIC Tibial Augment



Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.



Presence of latex in the product's composition: None

Material:

Implantable stainless steel (M30NW) according to standard ISO 5832-9

Manufacturing process:

Machining
Grit blasting
Polishing
Engraving
Inspection
Passivation
Final cleaning
Packaging
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2018

03

Technical information

Extension stem for Total Knee Prosthesis – Cemented



Manufacturer

AMPLITUDE, 11 Cours Jacques OFFENBACH
ZA MOZART 2, 26000 VALENCE, France.



Presence of latex in the product's composition: None

Material:

Implantable stainless steel (M30NW) according to standard ISO 5832-9

Manufacturing process:

Machining
Grit blasting
Engraving
Control
Passivation
Cleaning
Packaging
Packing
Sterilization
Release

CE Mark:

Notified body: BSI Group The Netherlands B.V. (CE2797)
Date EC marking first applied: 2003

04 Product identification

ANATOMIC posterior stabilized femoral component - cementless HA-coated

Implant engraving




non-contractual pictures

Information:

Implant size
Manufacturer Logo
C* / Batch number

*C = CoCrMo

Label

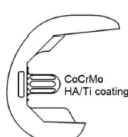		ANATOMIC® posterior stabilized femoral component <i>Condyle fémoral ANATOMIC® postéro-stabilisé</i> Cementless - HA coated <i>Sans ciment - HAP</i>		Size / Taille 1 Right / Droit	
					
REF	1-0204301	LOT	123456	EXP	2026-10-01
				EXP	2021-11-08
				QTY	1
 UDI (01)03701089512022(17)261001(11)211108(10)123456					
D-5.1					
					
 AMPLITUDE - 11 cours Jacques Offenbach - ZA Mozart 2 - 26000 VALENCE - FRANCE					

non-contractual pictures

Packaging

Paper box:

Dimension: 160x160x88 mm

Colour: dark green 354U

Primary packaging: blister

Secondary packaging: blister

04 Product identification

ANATOMIC posterior stabilized femoral component – cemented

Implant engraving




Information:

Implant size
Manufacturer Logo
C* / Batch number

*C = CoCrMo

non-contractual pictures

Label

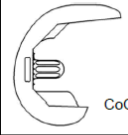		ANATOMIC® posterior stabilized femoral component <i>Condyle fémoral ANATOMIC® postéro-stabilisé</i> Cemented <i>A cimenter</i>		Size / Taille 1 Right / Droit
 CoCrMo				
REF 1-0204501	LOT 123456	2026-10-01	2021-11-08	QTY 1
 (01)03701089512206(17)261001(11)211108(10)123456				
D11D				
STERILE R                  2797				
AMPLITUDE - 11 cours Jacques Offenbach - ZA Mozart 2 - 26000 VALENCE - FRANCE				

non-contractual pictures

Packaging

Paper box:

Dimension: 160x160x88 mm

Colour: red 60

Primary packaging: blister

Secondary packaging: blister

04 Product identification

ANATOMIC tibial baseplate for fixed bearing insert - cementless HA-coated

Implant engraving



non-contractual pictures



Information:

Tibial baseplate:

Implant size
Manufacturer Logo
C* / Batch number


*C = CoCrMo

Standard keel:

Manufacturer Logo
S* / Batch number

*S = stainless steel

Label

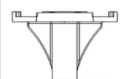		ANATOMIC® tibial base plate for fixed bearing insert <i>Embase tibiale ANATOMIC® pour insert fixe</i> Cementless - HA coated <i>Sans ciment - HAP</i>		Size / Taille 1	
 CoCrMo Stainless Steel HA/Ti coating					
REF	1-0204801	LOT	123456	2026-10-01	2021-11-08
QTY	1				
 (01)03701089512923(17)261001(11)211108(10)123456					
 STERILE R OR only MR MD Co CE 2797					
AMPLITUDE - 11 cours Jacques Offenbach - ZA Mozart 2 - 26000 VALENCE - FRANCE					

non-contractual pictures

Packaging

Paper box:

Dimension: 160x160x88 mm

Colour: lagoon green 100U

Primary packaging: blister

Secondary packaging: blister

04 Product identification

ANATOMIC tibial baseplate for fixed bearing insert – cemented

Implant engraving



non-contractual pictures



Information:

Tibial baseplate:

Implant size
Manufacturer Logo
C* / Batch number


*C = CoCrMo

Standard keel:

Manufacturer Logo
S* / Batch number

*S = stainless steel

Label

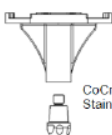		ANATOMIC® tibial base plate for <u>fixed</u> bearing insert		Size / Taille 1	
 CoCrMo Stainless Steel		<i>Embase tibiale ANATOMIC® pour insert fixe</i> Cemented <i>A cimenter</i>			
REF	1-0204901	LOT	123456	2026-10-01	2021-11-08
QTY	1				
 UDI (01)03701089513012(17)261001(11)211108(10)123456					
STERILE R CE 2797 AMPLITUDE - 11 cours Jacques Offenbach - ZA Mozart 2 - 26000 VALENCE - FRANCE					

non-contractual pictures

Packaging

Paper box:

Dimension: 160x160x88 mm

Colour: claret-coloured 385U

Primary packaging: blister

Secondary packaging: blister

04 Product identification

ANATOMIC fixed bearing insert posterior stabilized

Implant engraving



non-contractual pictures

Information:

Implant size
Batch number
UHMWPE
Manufacturer Logo

Label

		ANATOMIC® Fixed Bearing Insert <i>Insert fixe ANATOMIC®</i>		Size / Taille 8 Thickness / Epaisseur 20 mm	
 UHMWPE					
REF	1-0204785	LOT	123456	EXP	2026-10-01
				EXP	2021-11-08
				QTY	1
 UDI (01)03701089512909(17)261001(11)211108(10)123456					
D-3,4					
STERILE R             2797					
 AMPLITUDE - 11 cours Jacques Offenbach - ZA Mozart 2 - 26000 VALENCE - FRANCE					

non-contractual pictures

Packaging

Paper box:

Dimensions: 127x127x77 mm

Colour: yellow 84

Primary packaging: bag

Secondary packaging: blister

04 Product identification

Cemented resurfacing patellar implant

Implant engraving


Information:

Batch number
Manufacturer Logo



non-contractual pictures

Label

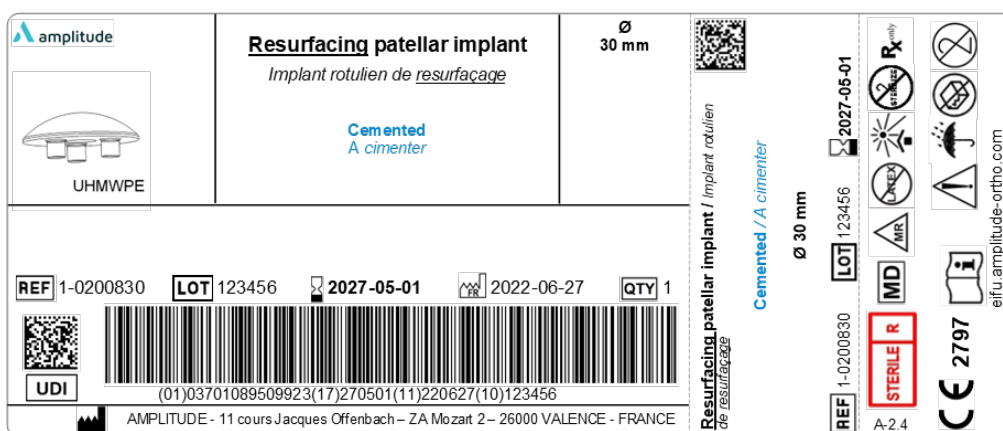

non-contractual pictures

Packaging

Paper box:

Dimensions: 110x70x25 mm

Colour: Sky / 80

Primary packaging: bag

Secondary packaging: bag

04 Product identification

Cemented Resurfacing Patellar Implant NM

Implant engraving


Information:

Batch number
Manufacturer Logo



non-contractual pictures

Label

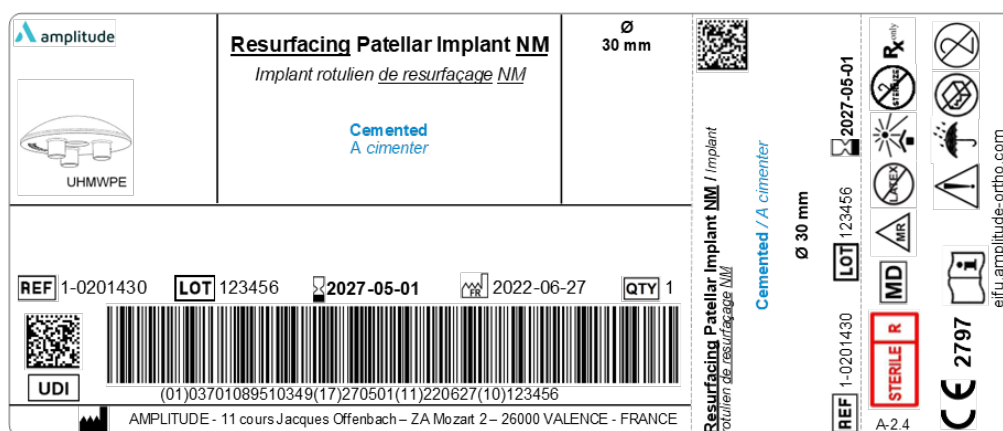

non-contractual pictures

Packaging

Paper box:

Dimensions: 110x70x25 mm

Colour: Vanilla 44

Primary packaging: bag

Secondary packaging: bag

04 Product identification

Cemented inset patellar implant

Implant engraving


Information:

Manufacturer Logo
Batch number



non-contractual pictures

Label

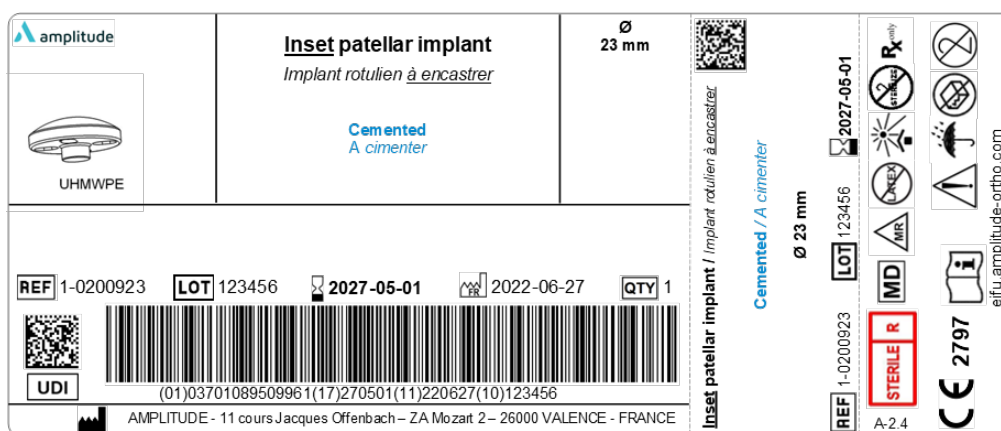

non-contractual pictures

Packaging

Paper box:

Dimensions: 110x70x25 mm

Colour: Fushia/64

Primary packaging: bag

Secondary packaging: bag


04 Product identification

Cemented Asymmetric congruent resurfacing patellar implant

Implant engraving

Information:

Manufacturer Logo
Batch number

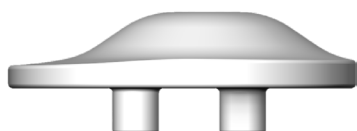

non-contractual pictures

Label

		Asymmetric Congruent Resurfacing Patellar Implant <i>Implant rotulien congruent asymétrique de resurfacement</i> Cemented <i>A cimenter</i>		23 x 30 mm 
 UHMWPE		23 x 30 mm		

REF 1-0201830	LOT 123456	2027-05-01	2022-06-27	QTY 1
---------------	------------	------------	------------	-------

 UDI	 (01)03701089510608(17)270501(11)220627(10)123456			
--	---	--	--	--

REF 1-0201830 A-2.4	 STERILE R	 MD	 MR	 No fire	 No open flame	 No heat	 No welding	 No X-ray	 No laser	 No ultrasound	 No magnetic field	 No electric field	 No static electricity	 No ionizing radiation	 No radioactive	 No biohazard	 No bioactive	 No bioresorbable	 No bioactive	 No bioresorbable	 No bioactive	 No bioresorbable	 No bioactive	 No bioresorbable	 No bioactive	 No bioresorbable	 No bioactive	 No bioresorbable	 No bioactive	 No bioresorbable	 No bioactive	 No bioresorbable
------------------------	--	---	---	--	--	--	---	---	---	--	---	--	--	--	---	---	---	---	---	---	---	---	---	---	---	---	---	---	---	---	--	---

AMPLITUDE - 11 cours Jacques Offenbach - ZA Mozart 2 - 26000 VALENCE - FRANCE

non-contractual pictures

Packaging

Paper box:

Dimensions: 110x70x25 mm

Colour: Brown 4735C/144


Primary packaging: bag

Secondary packaging: bag

04 Product identification

ANATOMIC Tibial Augment

Implant engraving

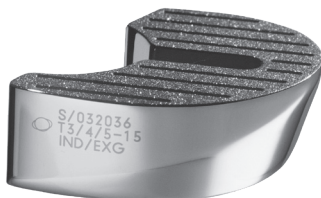


non-contractual pictures

Information:

Manufacturer Logo
S* / Batch number
Implant size, thickness and side

*S = stainless steel

Label

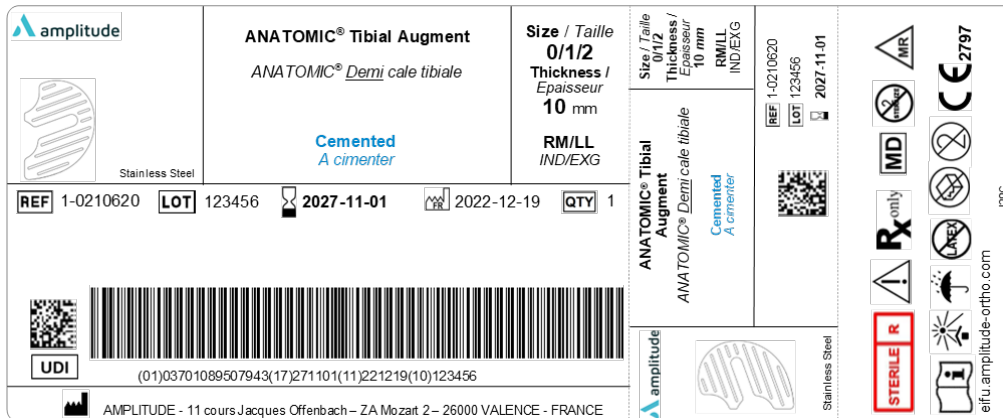

non-contractual pictures

Packaging

Paper box:

Dimensions: 115x90x30 mm

Colour: claret-coloured/385U

Primary packaging: blister

Secondary packaging: blister

04 Product identification

Extension stem for Total Knee Prosthesis – Cemented

Implant engraving




Information:

Manufacturer Logo
S* / Batch number

*S = stainless steel

non-contractual pictures

Label

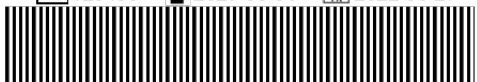		Extension stem for Total Knee Prosthesis <i>Quille longue pour Prothèse Totale de Genou</i>		Ø 10 mm Lg 75 mm	
Stainless Steel 		Cemented <i>A cimenter</i>			
REF 1-0200710	LOT 123456	2027-05-01	2022-06-27	QTY 1	
					
UDI		(01)03701089509695(17)270501(11)220627(10)123456			
 eifu.amplitude-ortho.com		171A			
STERILE R LATEX R only           MD CE 2797					
 AMPLITUDE - 11 cours Jacques Offenbach – ZA Mozart 2 – 26000 VALENCE - FRANCE					

non-contractual pictures

Packaging

Paper box:

Dimensions: 310x139x55 mm

Colour: yellow / 84

Primary packaging: bag

Secondary packaging: bag